Pediatric Masks – Instructions (1/6)



Materials:


- One piece of 100% cotton, tightly woven fabric.
 Cut to the size of 13" x 5.5" NO quilted or perforated fabric
- One piece of Cut Away Embroidery Stabilizer or Interface (5.5" x 5") *if available
 - Flannel can also be used
- 2 heavier <u>plastic-coated wire</u>, (such as a twist tie)
 4 6 inches in length (no paper twist ties as these will not stand up to washing)
- Extra wide double fold bias tape 8 inches per mask
- Braided elastic for ear loops. Recommend 1/16 to 1/8 inch, about 20 inches




www.bit.ly/face-masks-tutorial



- Wash and dry all fabric before sewing to preshrink.
- Cut material 5½" width by 13" length

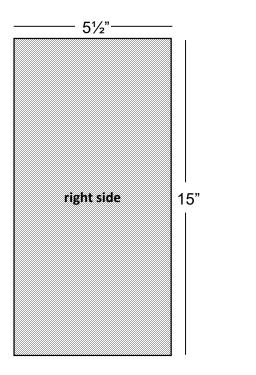


Iron top width of material down 1" at the top. Stitch filter and fabric at bottom right sides together ¼ in seam press and top stitch.

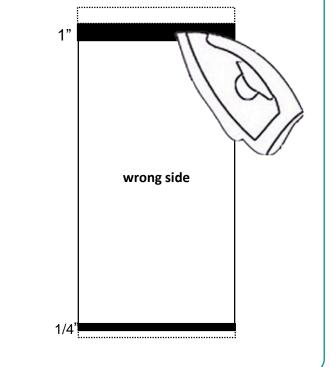


Pediatric Masks - Instructions (6




Fold under top ½" so the final top is ½" and then seam ¼" from the top of the material.

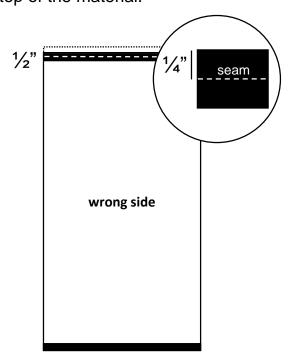

- Fold under bottom 1/4" so the final bottom is 1/4" and place washable filter material* (if available) in the seam and sew seam 1/4".
 - * Cut away embroidery stabilizer
 - * Interface

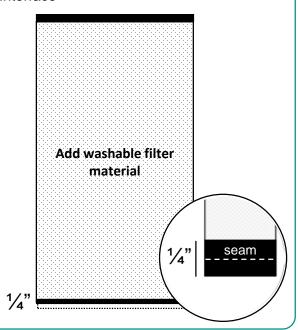


6 Cut wire 4" and insert so the wire touches the 1/4" seam on the top. Sew 1/4" enclosing the wire.

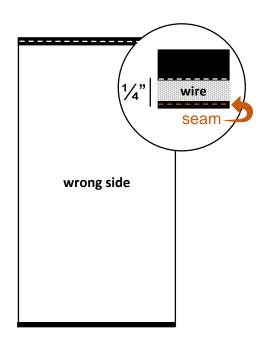

7 Fold the length of the material to the wrong sides of the fabric together so that bottom seam touches the top seam.

Zig zag together **1**" from each side. This leaves the middle open as a pocket on the back side of the mask.

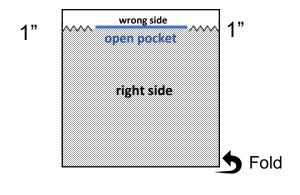


Pediatric Masks – Instructions (3/6)



8

Iron 3 pleats on the right side of the mask and press. The first pleat should be $1\frac{1}{2}$ " from the top of the mask. The second pleat about $\frac{1}{2}$ ", third pleat $\frac{1}{2}$ " and remainder of the fabric $\frac{1}{2}$ ". The length should be 3".


Sew 1/4" on each side of the mask to hold in the pleats.

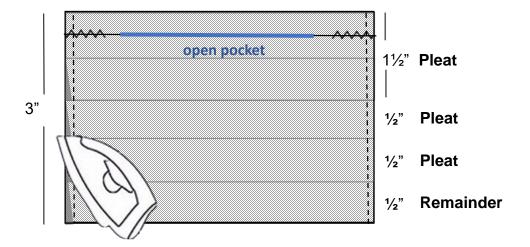

9 Cut a piece of fabric 2" wide by 4" long for the bind. With right sides together on the pocket side of the mask, extend ½" of binding at the top of the mask and ½" (or remainder of the binding) extended on the bottom of the mask.

Fold this ½" towards the non-pocket side of the mask on both the top and bottom of the mask.

Sew ½" seam to incorporate the folded binding at the top and bottom.

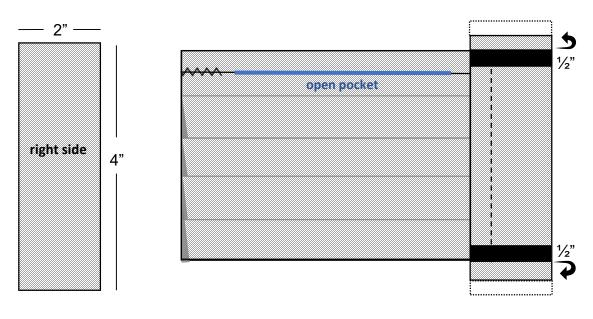



Pediatric Masks – Instructions 4/6)




10

Unfold the seamed binding to the side of the mask and fold under ½" toward the front of the mask (non-pocket) and **fold again** over the seam and top stitch along the ½" stitch.

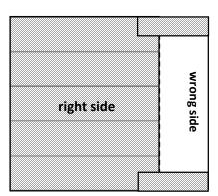

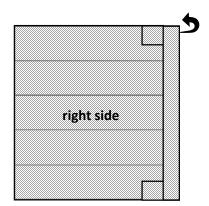

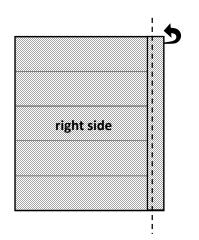

11

Cut two 6" pieces of 1/8" elastic for the ears. Thread elastic through the $\frac{1}{2}$ " pocket created by the previous step.

Tie ends in a knot and pull knot back through to hide inside the pocket *Make sure to stitch so they don't come out



Mask (front)

Pediatric Masks – Instructions (5/6)





Mask (back)



Mask (pocket open)

Pediatric Masks – Instructions (6/6)





Filter Sewn In (back)



Filter Sewn In (Front)